

WatchPod Cleaning Instructions



All of the surfaces listed below will withstand cleaning using a solution of 0.5 Hydrogen Peroxide (such as Virox®) or a 70% Isopropyl alcohol solution following the recommended cleaning frequency of a patient room environment as set out in such guidelines as Best Practices for Environmental Cleaning by PIDAC. When cleaning with either of the above solutions it is a requirement that the solution not be allowed to sit for an extended time on the surface, but should be wiped thoroughly within a few moments of application.

Bed Frame/Bedside Cabinet (Low Pressure Laminate)

Dust with a soft, clean cloth. To remove fingerprints etc. scrub with a damp soapy cloth followed by a rinse with cold water. Stubborn stains can be removed with hot water and a bit of vinegar. Avoid using abrasive cleaning tools as these may scratch the laminate surface. It may be necessary to use a mild non-abrasive cleaner.

Wall Panels (High Pressure Laminate)

Branded Cleaner and Sanitizer Resistance for Wilsonart® Laminate. No effect was exhibited except as noted (* or **) on the following:

1. Clorox Healthcare Bleach Germicidal Cleaner *
2. Clorox Healthcare Versa Sure Cleaner Disinfectant Wipes
3. Oxivir TB
4. Oxivir 1
5. Virex II 256
6. Benefect
7. PDI Super Sani-Cloth Germicidal Disposable Wipes
8. PDI Sani-Prime Germicidal Spray
9. Expose II 256
10. Stride Floral Neutral Cleaner *
11. PURELL Advanced Instant Hand Sanitizer

*Causes slight change of gloss or color. ** Causes slight damage, with degree of damage proportionate to length of exposure and concentration.



HEAD OFFICE: Level 1, 1 Queens Road, Melbourne VIC 3004
MELBOURNE: 62 Barrie Rd, Tullamarine VIC 3043
SYDNEY: 1 Bearing Rd, Seven Hills NSW 2147
BRISBANE: 4/7 Lathe St, Virginia QLD 4014
ADELAIDE: 60 Wing St, Wingfield SA 5013
PERTH: 14 Somersby Rd, Welshpool WA 6106